


# DIAPER/BRIEF SWAB COLLECTION AND TRANSPORT INSTRUCTIONS

IT IS IMPORTANT THAT THE BRIEF OR DIAPER DOES NOT CONTAIN ANY FECAL MATERIAL AND/OR BLOOD IN IT. SAMPLES WITH VISIBLE FECAL MATERIAL/BLOOD WILL BE REJECTED.

**Important Note:** Wash hands and wear personal protective equipment (i.e., lab coat, gloves, etc.).

## DIAPER/BRIEF STRUCTURE

Topsheets will be manually removed to expose the "Acquisition/Distribution Layer" for swabbing.

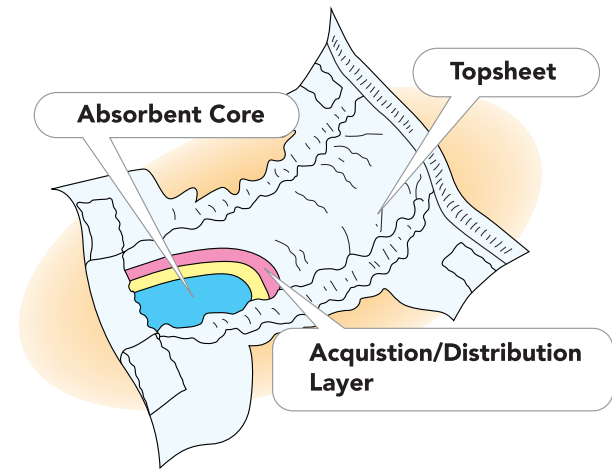

ALL SPECIMEN CONTAINERS MUST BE LABELED WITH BOTH THE PATIENT'S FIRST AND LAST NAMES AS WELL AS A SECOND IDENTIFIER SUCH AS THE PATIENT'S MEDICAL RECORD NUMBER OR DATE OF BIRTH. CONTAINERS WITHOUT TWO IDENTIFIERS WILL BE REJECTED.

### STEP 1



Ensure that the brief or diaper does not have any fecal material or blood on it. Cut open the diaper in the center to reveal the "Acquisition/Distribution Layer". DO NOT cut through this layer.

### STEP 4



Open the swab tube. Set the cap down with the inside facing up. Add the swab to the media.

### STEP 2



Open the eSwab package and remove the swab using the opposite end of the swab tip. DO NOT touch the end of the swab stick, to limit contamination. Set the tube to the side.

### STEP 5



Place the swab tip into the opened tube. Push on the back end of the swab stick so the swab tip comes off. Put the cap back on the tube.

### STEP 3



Place the swab tip onto the brief or the diaper absorbent material and press down. Rotate the swab 2-3 times to ensure the swab is saturated with Urine.

### STEP 6



Label specimen tube with Patient Name, DOB, and Collection Date. Package and send to laboratory.